

# Application Notes

## Impulse 6000D/7000DP Ventricular Fibrillation Amplitudes

### INTRODUCTION

Fluke has made some changes to the Impulse 6000D/7000DP ECG amplitudes to improve customer satisfaction with the instrument. This paper describes the implications for the VFib waves.

### REFERENCE LEAD

The reference lead is the lead that matches the ECG Amplitude setting. For example, with a Lead II reference, at a setting of 1 mV Amplitude, Lead II is 1 mV.

All other leads will be a defined percentage of the reference lead.

### FIRMWARE CHANGE

Firmware changed from version 1.02 to 1.03.

For firmware 1.02 and earlier, Lead I was the ECG reference lead.

For firmware 1.03 and later, you can select either lead I or lead II to be the reference lead, but Lead II is the default selection. The selection is stored in non-volatile memory.

### HARDWARE ECO (Engineering Change Order)

The defib paddle ECG amplitude is always proportional to Lead II.

Originally, the defib paddle ECG amplitude was 40% of Lead II. The ECO changed 2 resistors in the circuit to make it equal to Lead II.

### ECG AMPLITUDES

Normal Sinus wave ECG amplitudes, as a percentage of reference lead setting:

Reference Lead	I	II	III	Defib Paddles with ECO	Defib Paddles before ECO
II	70	100	30	100	40
I	100	150	50	150	60

### Fluke Biomedical

6045 Cochran Road, Cleveland, OH 44139-3303 U.S.A.  
 Tel: 440.248.9300, Toll free: 800.850.4608, Fax: 440.349.2307  
 Email: sales@flukebiomedical.com  
[www.flukebiomedical.com](http://www.flukebiomedical.com)

# Application Notes

## VENTRICULAR FIBRILLATION (VFIB)

VFib is an important wave needed by Automated External Defibrillators (AEDs). It is an irregular wave that varies throughout but its overall peak to peak amplitude is proportional to a Normal Sinus wave. Fine VFib is 1/2 the amplitude of Coarse VFib:

Wave	Percentage of Normal Sinus
Vfib Coarse	85
VFib Fine	42

Therefore, the amplitude of VFib Coarse as a percentage of Reference is:

Reference Lead	I	II	III	Defib Paddles with ECO	Defib Paddles before ECO
II	60	85	25	85	34
I	85	128	42	128	51

And the amplitude of VFib Fine as percentage of Reference Lead is:

Reference Lead	I	II	III	Defib Paddles with ECO	Defib Paddles before ECO
II	30	42	12	42	17
I	42	64	21	64	26

To get an AED to recognize VFib, the user should be aware of these amplitudes.

If the amplitude is too small, the AED might not get enough signal to recognize it.

If the amplitude is too large, the AED might think the VFib transitions are beats instead of fibrillation.

The user can adjust the ECG Amplitude setting as needed to get the AED to recognize the VFib.

**Fluke Biomedical**

6045 Cochran Road, Cleveland, OH 44139-3303 U.S.A.  
 Tel: 440.248.9300, Toll free: 800.850.4608, Fax: 440.349.2307  
 Email: sales@flukebiomedical.com  
[www.flukebiomedical.com](http://www.flukebiomedical.com)